UriSet 24 The Complete Set for Standardized 24h Urine Collection



- Standardization of preanalytical variables
- Efficient sample management
- Optimal patient comfort



UriSet 24

UriSet 24, an innovative product for 24 h urine collection, is ready for immediate use and relieves users from the formerly time-consuming compilation of individual components. Another inherently new aspect is that only the 30 ml transport tube, rather than the full container, is transported to the doctor or laboratory. These product features, in unison with the standardized stabilizer, are convincing benefits for preanalytics, laboratory work and patients.

Preanalytics

Standardized collection, transport and storage of 24 h urines

Laboratory

Economic sample management: Long-term stability enables sorting and batch processing

Patient

Optimal patient comfort facilitates urine collection



The all-round convincing solution

1 Optimal handling

Container with extra large opening (80 mm) and ergonomically moulded handle.

3,000 ml filling volume

Accounts for 91% of all urine volumes recorded. Conventional containers can only hold approximately 60%.

Superior readability 3

Due to a transparent inspection strip and moulded graduation in 100 ml intervals.

Transport tubes with UV light protection

Serve as a primary receptacle for transport packaging in conformity with the P650 packaging regulations. The white coloration of the transport tube provides optimum UV light protection to ensure increased specimen stability. Small sample volume keeps any unpleasant odours to a minimum. Reduces laboratory workload.

Standardized 5

Hydrochloric acid volume and concentration enables long-term sample storage without refrigeration for up to 7 days.

Simultaneous stabilization

of various diagnostically relevant marker molecules: catecholamines, metanephrines, vanillin phenylic acid, homovanillic acid and 5 hydroxyindole acetic acid. Descriptive literature is available on request.

500 ml receptacle 6

> Particularly convenient for children and female patients because the urine can be passed into the receptacle rather than into the container.

Safe transfer

The receptacle enables safe transfer of the urine from the container into the transport tube.

The flexible receptacle can be compressed to form a thin-shaped pouring neck to avoid spills.






















The complete set for standardized and patient-friendly 24h urine collection

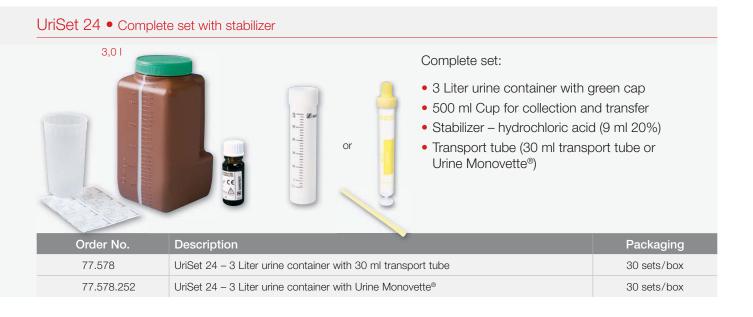



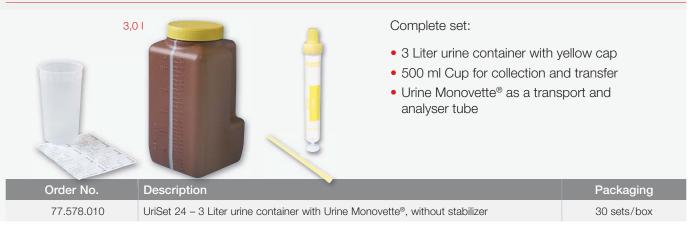

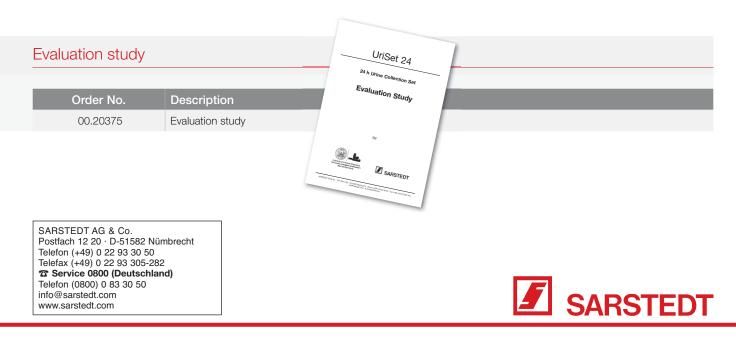

366a-200-1104-0210